

SOCKIT[®]GEL

PATIENT INSTRUCTION SHEET

- ✔ Remove resealable cap, apply the gel to the affected area, and reseal the tip by snapping into place.



- ✔ Start using SockIt Gel the day of your procedure. Apply as often as needed for pain relief, but at least 4 times each day until the syringe is empty for optimal healing.
- ✔ Do not rinse with salt solution or any other product, as this will rinse the gel out.
- ✔ SockIt Gel contains no drugs, and is non-toxic and safe when swallowed.
- ✔ Store SockIt Gel at room temperature (between 50° - 86° F); avoid excessive heat or freezing. Do not store in direct sunlight.
- ✔ If you have any questions regarding the use of SockIt Gel, please call 800.755.0044 or visit www.sockitgel.com.

COMPLETE YOUR PROFESSIONAL
CLEANING WITH ORASOOTHE!



PROTECT • RELIEVE • HEAL

ASK YOUR HYGIENIST TODAY!



800.755.0044 • www.orasoothe.com

HOW IS YOUR MOUTH?

DISCOMFORT OR PAIN FROM
CANCER
TREATMENT?

ASK FOR ORASOOTHE TODAY!

"OraSoothe always takes care of my mouth sores from chemo within several days, and I definitely see a difference after taking it even one time. This product is my 'go to' when I need healing and relief immediately"
-Catherine L, TX



www.orasoothe.com • 800.755.0044