

# ORASOOTHE<sup>®</sup>

“SOCKIT” GEL

## PATIENT INSTRUCTION SHEET

- Remove resealable cap, apply the gel to the affected area, and reseal the tip by snapping into place.



- Start using OraSoothe SockIt Gel the day of procedure. Apply as often as needed for pain relief, but at least 4 times each day until the syringe is empty.
- Do not rinse with salt solution or any other product, as this will rinse the gel out.
- OraSoothe SockIt Gel contains no drugs, and is non-toxic and safe when swallowed.
- Store OraSoothe SockIt Gel at room temperature (between 50°- 86° F); avoid excessive heat or freezing. Do not store in direct sunlight.
- If you have any questions regarding the use of SockIt Gel, please call 800.755.0044 or visit [www.sockitgel.com](http://www.sockitgel.com).



COMPLETE YOUR PROFESSIONAL  
CLEANING WITH ORASOOthe!

**PROTECT • RELIEVE • HEAL**

**ORASOOthe**<sup>®</sup>  
oral coating rinse

*(oral hydrogel wound dressing)*

**ASK YOUR HYGIENIST TODAY!**

**HOW IS YOUR MOUTH?**

**DISCOMFORT OR PAIN FROM**

**CANCER  
TREATMENT?**

**ASK FOR ORASOOthe TODAY!**

"OraSoothe always takes care of my mouth sores from chemo within several days, and I definitely see a difference after taking it even one time. This product is my 'go to' when I need healing and relief immediately"  
-Catherine L, TX



[www.orasoothe.com](http://www.orasoothe.com) • 800.755.0044